

## TRIBUstrip



**EAN Code:** 8595635303593

**Catalog number:** Ts0300

**Package size:** 300 tests

**Storage:** 2-8 °C

**Producer:** TestLine Clinical Diagnostics s.r.o.

### Description:

- The test principle is an enzyme hydrolysis of tributyrin. This reaction causes colour change of acidobasic indicator.
- The kit enables 300 pcs.
- Shelf life: 24 months.

### Application:

- Microbiological diagnostic test for differentiation between *Moraxella* (*Branhamella*) and *Neisseria* genus.

### Brief assay procedure:

1. Throw one tributyrin strip into the suspension of tested strain in 1 ml of buffered saline.
2. Incubate the tested sample at 37 °C (without CO<sub>2</sub>).
3. Read the final results after 18-20 h.
4. Evaluate results.